



Thermo Scientific CW3 Cell Washer

Fits in. Stands Out.

Thermo Scientific™ CW3 Cell Washer

Precise, automated performance designed to provide thorough, reproducible blood cell washing, up to 24 tubes per run.



Fits in.

- To your space, with its compact footprint
- To your processes and protocols, with flexibility of 12- or 24-place rotors and standardized, repeatable procedures

Stands out.

- Fully automated cell washing cycle saves time
- User-friendly operation with keypad control and easy-to-read digital displays
- Automatic alerts indicate if the saline level is low, and when a run is complete

Easy-to-read digital display for simple, user-friendly operation


Condition-setting panel allows more flexible protocols to optimize washing performance



SUBSEQUENT APPLICATIONS

Antiglobulin tests, ABO compatibility, Rh testing, Cross matching and Antibody screening.

Thermo Scientific CW3 Cell Washer

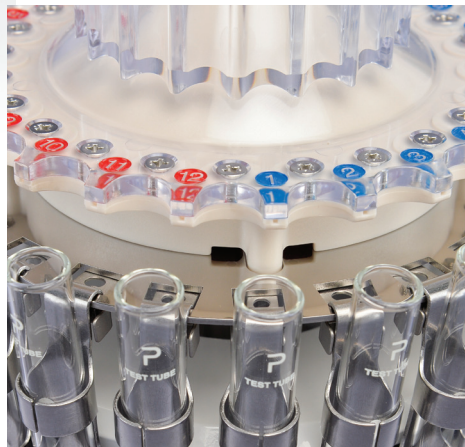

Efficient Performance

- Fast run – finish washing 24 samples in 3 minutes
 - ▶ Achieve optimal corpuscle dryness by setting desired decanting speed
 - ▶ Improve corpuscle washing with optional overflow method during centrifugation step
- Optimize pelleting performance
 - ▶ For washing, tubes swing at a fixed positive angle to sediment cells rapidly at the bottom of the tube
 - ▶ For decanting, the rotor holds tubes at a slightly negative angle and the saline is decanted centrifugally


User-Friendly Design

- Save time with pre-set programs in automatic mode
- Manual mode available to wash the cells step-by-step, allowing closed monitoring of washing process
- Tube racks with red and blue number labeling for easy sample balancing
- Cleaning convenience with easy-to-remove catch basin, rotor and rotating bowl

Safe Operation

- Automatic alerts for low or disrupted saline level and for a completed run
- View port on top of unit is available for speed calibration
- Reduces risk of cross-contamination with the saline distributor mounted directly on top of the rotor
- Low-maintenance design with easy-to-change pump tubing

Operational Sequence

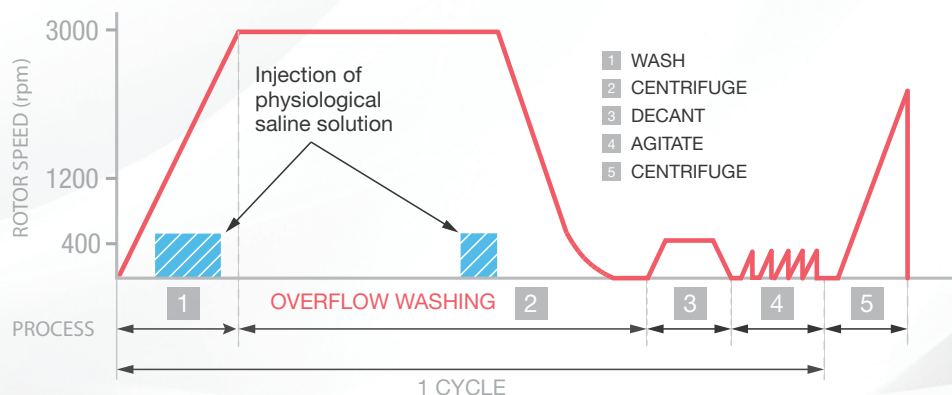



Thermo Scientific CW3 Cell Washer

Specifications	Description
Maximum Speed	Centrifuge 3,000 rpm Wash 1,200 rpm Decant 350-500 rpm
Maximum RCF	Centrifuge 1,180 x g Wash 190 x g
Maximum Capacity	24 tubes
Tube Dimensions	12 x 75 mm or 10 x 75 mm
Process Display	4-stage display: Wash ■ Centrifuge ■ Decant ■ Agitate
Process Time Per Cycle	60 seconds (Automatic centrifugation time: 35 seconds)
Timer	Automatic: 1 to 99 seconds in 1-second increments Manual: 1 to 999 seconds in 1-second increments
Cycle	1-9 cycle settings
End-of-Run Notification	No sound ■ 5 selectable melodies ■ Electronic beeps
Safety Devices	Door interlock ■ Dual-overspeed detector ■ Saline level sensor
Dimensions (W x D x H)	37 x 45 x 41 cm ■ 14.6 x 17.7 x 16.1 in
Weight	28 kg ■ 61.6 lb
Standards	IEC 61010-1 ■ IEC 61010-2-020 ■ EN ISO 13485
Certifications	UL listed ■ CSA certified ■ CE marked ■ FDA Class 2 ■ IVD compliant

Ordering Information

Cat. No.	Description
75007404	CW3 cell washer, 120 V, 60 Hz
75007402	CW3 cell washer, 120 V, 60 Hz, with 24-place rotor assembly with distributor
75007400	CW3 cell washer, 120 V, 60 Hz, with 12-place rotor assembly with distributor
75007405	CW3 cell washer, 220-230 V, 50/60 Hz
75007406	CW3 cell washer, 220-230 V, 50/60 Hz, with 24-place rotor assembly with distributor
75007407	CW3 cell washer, 220-230 V, 50/60 Hz with 12-place rotor assembly with distributor
75000020	24-place rotor assembly with distributor
75000021	12-place rotor assembly with distributor

Rx Caution: Federal law restricts this device to sale by or on order with a qualified clinical facility manager or equivalent.

Find out more at thermofisher.com/cellwasher

Australia +61 39757 4300
Austria +43 1 801 40 0
Belgium +32 9 272 54 82
China +800 810 5118 or
 +400 650 5118
France +33 2 2803 2180
Germany national toll free 0800 1 536 376
Germany international +49 6184 90 6000
India toll free 1800 22 8374

India +91 22 6716 2200
Italy +39 02 95059 552
Japan +81 3 5826 1616
Korea +82 2 2023 0600
Netherlands +31 76 579 55 55
New Zealand +64 9 980 6700
Nordic/Baltic/GIS countries
 +358 10 329 2200
Russia +7 812 703 42 15 or
 +7 495 739 76 41

Singapore +82 2 3420 8700
Spain/Portugal +34 93 223 09 18
Switzerland +41 44 454 12 12
UK/Ireland +44 870 609 9203
USA/Canada +1 866 984 3766

Other Asian countries +852 3107 7600
Countries not listed +49 6184 90 6000

ThermoFisher
 S C I E N T I F I C